

BIOCARD PRO COVID-19 RAPID ANTIGEN


Immediate on-site Covid 19 Antigen Testing

- Based on ColloidalGold Chromatographic Immunoassay.
- Rapid Qualitative detection of Covid 19Antigen.
- Easyprocedure & Fast Test Time (7~10minutes).
- Sample Type:Human Nasopharyngeal Swab.
- All Material provided in Kits to perform assay/ No Equipm ent required.
- Room Temperature Storage (4-30°C).



BIOCARD Pro **COVID-19 Rapid Antigen**


BIOCARD Pro COVID-19 Rapid Antigen kit is a chromatographic immunoassay for the qualitative detection of specific antigens to COVID-19 present in human nasopharyngeal swab

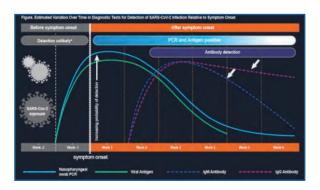

Assay Procedure



Add 15to 20 drops of Sample Extraction Buffer

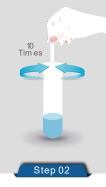

Insert the patient swab sample into the extraction solution tube. Then, swirl swab at least 10 tim es



Remove the swab while squeezing the liquid from the swab by pressing tube side



Coverthe tube with a filter cap and tighten the lid



Incubate the tube for 1-2minutes



Add 2 drops of the extraction solution with specimen into the sample well

Read results after 7~10minutes Interpretation of Results





Invalid

Negative *Please read result within 15Min

Specifications

| Method | :Immuno Chromatography | Assay Tim e |
|---------------|------------------------|---------------|
| Certification | : ICMR Approved | Shelf Life |
| Sample Type | :Nasopharyngeal Swab | Storage Tempe |

erature

 $: 7 \sim 10$ Minutes :24 Months :4-30°C

*Please note: Product specifications are subject to change without prior notice owing to product modifications, improvements / up-gradation.

Marketed and Exported : **SRI LAKSHMI PHARMA** 2-4-89/1, Enugula Bazar Narasaraopet-India. Contact:+91 98481 77013 Email: srilakshmipharma.dsrao@gmail.com

